INSTRUCTIONS FOR USE

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EN

SAFILENS CONTACT LENSES – INSTRUCTIONS FOR USE

This leaflet contains important information, and recommendations on the proper use of the product.

Please read it with care and retain for future reference. It is also essential to comply with the instructions of your own eye care professional as well as those indicated on the package.

INTENDED USE:

The contact lenses are intended, for the optical correction of ametropias and where specified presbyopia.

RECOMMENDED USAGE:

It is recommended to change your contact lenses daily.

HANDLING THE LENSES:

Wash and dry your hands carefully before handling the lenses, and make sure that they are not inside out; never use the lenses if they have been damaged in any way.

LENS INSERTION:

Do not apply make-up until you have inserted your lenses. To avoid mixing up your lenses, insert them one by one. Make sure the lens is not inside out before insertion (see figures 1 & 2 for ref.).

Place the lens on your index finger to verify shape. Using your other hand's index finger and second finger hold up the upper lid and hold down the lower lid. Place the lens on the eye (see figure 3).

Repeat this step for the other eye.

For any doubt please contact your eye specialist.

LENS REMOVAL:

Look up and hold down the lower lid. Using your index fingertip and thumb tip to gently pinch the lens off the eye (see figure 4). Repeat this step for the other eye. For any doubt please contact your eye specialist.

THERAPEUTIC USE:

Contact lenses without refractive power (0.00D) might be also indicated for therapeutic use. Use as a bandage contact lens for corneal protection and corneal pain relief during treatment of ocular pathologies as well as post-surgical conditions. Strictly follow your specialist instructions for the therapeutic use.

UV BLOCK:

Contact lenses including UV protection may not replace sunglasses because they do not completely cover the entire eye anterior segment. Therefore, contact lenses wearers must continue to bring sunglasses when exposed to UV rays.

PRECAUTIONS:

Never store the lenses in a saline solution because it does not protect them from harmful contaminations; do not re-use a lens; do not use the products beyond their use-by date.

Use cosmetics, soaps, creams and aerosols with care because they can irritate the eyes.

Smoke increases the risk of ulcerative keratitis. Inform your employer that you wear contact lenses. In the event of any eye discomfort, remove the contact lenses and see your specialist.

Do not sleep wearing contact lenses because this poses the risk of keratitis. Keep out of children's reach.

CONTRAINDICATIONS:

Do not use the lenses in any of the following situations:

infections, inflammation, allergies or any disorders affecting the cornea, conjunctiva or eyelids, ocular dryness, damaged lenses, dirty lenses, sensation of irritation or pain in the eye.

Safilens S.r.l.

Via Maris Grazia Deledda 5,34079

Staranzano Italy

Importer: Einit LTD

Communication Center, Neve Eilan

TEL: 02-5338810

With approval from the Ministry of Health 9530405

*The leaflet is written in male language for convenience only, but everything stated in this leaflet is intended for both women and men.

Fig. 1.

Fig. 2.

Fig. 3.

IN

Fig. 4.

OUT

Safilens S.r.l.
Via Maris Grazia Deledda, 5
IT34079 Staranzano ITALY

CE 0123

STERILE

MD

SYMBOLS

	= Sterilized using steam or dry heat
	= Use-by date
	= Spherical power
	= Base curve
	= Diameter
	= Do not re-use
	= Do not use if packaging is damaged
	= Medical Device
	= Read instructions before use

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